# Predictor

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The Predictor pregnancy test is for self-testing. Please read the entire instructions carefully before use. Do not open the foil pouch until you are ready to perform the test. Have a watch, clock or timer available.

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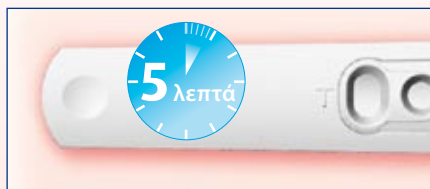
### HOW TO PERFORM THE TEST

Predictor can be used at any time of the day, from the day your period is due. Take the pregnancy test out of the foil pouch. Open the test by removing the cap from the stick.

Hold the urine absorber downward in your urine stream for at least 5 seconds until it is wet. Do not invert the stick afterwards, but keep it pointing downwards.

Replace the cap back on the stick. Lay the stick down flat with the windows facing you and wait 5 minutes before reading the result.

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### HOW TO READ THE RESULT

After you have completed the test, whether you are pregnant or not, a pink/purple line will appear in the round control window. This indicates that the test is working properly.

Read the result 5 minutes after you have conducted the test. Do not interpret the result after 30 minutes.

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**You can conclude you are pregnant** if a pink/purple line appears in the large oval test window-(T). Even if the line is very pale, it still means you are pregnant. A positive result may in some cases be observed in less than 1 minute.

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**You can conclude you are not pregnant** if no pink/purple line appears in the large oval test window-(T).

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**Important:** There should always be a control line visible. If there is no control line, the test has not worked properly and the test result is invalid. There might have been insufficient urine on the absorber. Repeat the test with a new device, following the instructions carefully.

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## POSITIVE RESULT? KEEP THE TEST!

A positive test result remains positive over time. You can keep your positive test result if you want. In order to make this more practical for you, Predictor has optimized its design: one click, and the handle is removed. This makes it more compact to store or to put in your baby's first album!

## COMMON QUESTIONS ABOUT THE PREDICTOR PREGNANCY TEST

### How accurate is Predictor?

Predictor has been clinically proven to be over 99% accurate, as reliable as any pregnancy test performed by a doctor.

### How does Predictor work?

When a woman becomes pregnant, her body starts to produce a special hormone known as hCG (human Chorionic Gonadotrophin), which is present in the urine and increases daily. Predictor is uniquely designed to detect already very small amounts of this pregnancy hormone in your urine. You can test from the day that your period is due.

### How soon can I use Predictor?

Predictor can detect hCG in your urine with over 99% accuracy as early as the day your period is due. If you have irregular periods, to work out when to test, take your longest cycle and add one day, to avoid testing too early.

Testing earlier than the day your period is due is possible, but not recommended, as the result will not be over 99% reliable. If you test earlier, you might be pregnant, but still not have enough hCG produced to be detected by the test. On the other hand, an early positive test result is not reliable either, because pregnancies in an early phase (i.e. detected before your period is due) can end in a natural way.

### Do I have to test in the morning?

No, you can use Predictor any time of the day. Excess fluid intake immediately before testing may dilute the pregnancy hormone so drink normally before you test.

### After 5 minutes, the test window had no line, but some minutes later it appeared.

If the line appears in the test window within a time frame of 30 minutes, the result is positive; no matter how pale the line is.

### How long will my result remain before it fades away or changes?

A positive result will remain unchanged, however the background may become darker. You can keep the test with the positive result if you want.

On the other hand, with a negative result, a very faint line may appear in the test window after more than 30 minutes. This color deposit, showing after 30 minutes, is not a positive result. Remember: do not interpret the result after 30 minutes.

### Can I still use the test if I have collected a urine sample in a collection cup?

Yes, as long as the urine sample is in a clean, dry container. Do not use detergent or solvent to clean it. Test right away: immerse the urine absorber in the urine for at least 10 seconds. Pay attention that the urine level does not go above the plastic housing of the stick. Wait for five minutes before you read the result.

### What do I do if I get a positive result?

If the test is positive, we recommend you consult a doctor to discuss your pregnancy and what steps and precautions you should take next. Early prenatal care is very important to ensure the health of both you and your baby.

### The test result says I am not pregnant, but I am still not sure?

If the result says you are not pregnant, no pregnancy hormone (human Chorionic Gonadotropin) has been detected. You may have miscalculated the day your period was due. If your period has not started within a week, repeat the test using another Predictor home pregnancy test. If the result is still negative and you still have not started your menstruation, consult your doctor.

### Can any medication affect the result?

Predictor is only affected by fertility drugs which contain hCG (human Chorionic Gonadotropin) or hMG (human Menopausal Gonadotropin). Other fertility drugs (such as Clomid), antibiotics, painkillers or the contraceptive pill, should not affect the result.

### Can I use the Predictor test more than once?

No, you can use each test only once.

Not pregnant yet, but wanting to be? Predictor Ovulation test indicates your 2 most fertile days of your cycle.

For more information see [www.omega-pharma.gr](http://www.omega-pharma.gr)

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## ANY FURTHER QUESTIONS?

Mail to: [contact@omega-pharma.gr](mailto:contact@omega-pharma.gr)

Write to: 19th km Nat.Road - Athens-Lamia - 14671 N.ERYTHREA-GREECE

In correspondence, please quote the batch number as printed on the pack. Or call the Predictor helpline (210 81 88 942/6946 79 19 18) between 9h and 17h.

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Composition: test stick containing coated membrane with monoclonal and polyclonal antibodies and a pad containing mouse monoclonal IgG (antibody) dye conjugate in protein matrix.

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Store at room temperature (15°C - 28°C) in a sealed pouch. If foil is damaged, do not use the test.

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